SHOFU TEMPORARY CEMENT SOFT	
EN Instructions for use IT Istruzioni per l'uso EL Οδηγίες χρήσης	
DE Gebrauchsanleitung RU Инструкция по применению PT Instruções de uso	
FR Mode d'emploi PL Instrukcja obsługi ZH 使用说明书	
NL Gebruiksaanwijzing RO Instrucțiuni de utilizare	
ES Instrucciones de uso SV Bruksanvisning	
	03:2019-03
	71342
Manufacturer	
SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983,Japan www.shofu.com	EC REP SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen,Germany
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059,USA	SHOFU DENTAL ASIA-PACIFIC PTE.LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
CE 0123	

EN	READ CAREFULLY BEFORE USE
-----------	----------------------------------

INDICATIONS

- Temporary cementation of temporary crowns, etc.
- Indirect pulp capping, temporary sealing after cavity preparation
- Temporary cementation of deciduous cavities

PRECAUTIONS

- Do not use this product on patients with known allergies to this product or polycarboxylate cement.
- Operators with known allergies to this product or polycarboxylate cement must not use this product.
- If any inflammation or other allergic reactions occur on either patient or operator, immediately discontinue use and seek medical advice.
- The liquid and mixture of this product should not come in contact with intraoral soft tissue, skin or eyes. In case of accidental contact with intraoral soft tissue or skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- This product contains free silica. Inhalation of free silica for long periods could cause damage to the lung. Use local dust extractor, dust protective mask, etc to avoid harmful influence of the dust on the human body.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Standard Powder/Liquid Ratio: 2.2 g/1.0 g

Dispense one level scoop of powder onto a mixing pad using the measuring scoop provided. Then, dispense two drops of liquid separately.
- Mixing

Divide dispensed powder into 2 equal portions; introduce first half to the dispensed liquid and mix with a spatula within about 15 - 20 seconds. Then, incorporate the remaining half gradually and mix the whole until it becomes homogeneous paste. Mixing must be completed within about 45 seconds.
- Temporary cementation/sealing

Immediately use the mixture for cementation, sealing or lining.
- Setting and removal of excess material

After cementation or filling, keep the area from being contaminated with saliva for about 8 minutes. Then, remove excess hardened material following the conventional method.
- Removal

Remove provisional restoration and residues of this material. Thoroughly clean the tooth surface for permanent restoration.

NOTE ON USE

- Always use the measuring scoop provided. Level off the powder with built-in leveler for accurate dispensing.
- Invert the liquid bottle prior to use to dispense drops of bubble-free liquid.
- Remove any hardened liquid plugging the opening of the nozzle with wet gauze etc. for accurate measuring.
- Tightly close the bottle caps (Powder and Liquid) immediately after each use. Use extra caution to keep the powder away from any moisture.
- Tooth surface to be cemented or filled should be kept slightly moist. Do not desiccate the surface.
- In the case where pulp is exposed or the affected area is close to the pulp, cap the pulp with pulp capping agent (calcium hydrate etc).
- Do not use for cementation of permanent restoration.
- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperature and high humidity. Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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