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EN

DIRECTIONS FOR USE

Select optimal Opaque shades according to the color chart and dispense each powder in small amount on a glass slab. Then, mix with VINTAGE Halo OPAQUE LIQUID. Apply the mixture with a nylon brush, lightly condense and fire.

- Optimal consistency of the mixture (for 2 firings)
 - 1st firing Mix generous amount of opaque liquid within the extent that the mixture is homogenous to obtain low consistency.
 - 2nd firing Optimally consistent mixture spreads slowly on a glass slab immediately after it is collected in one lump with a spatula.

NOTE ON USE

1. Always use VINTAGE Halo OPAQUE LIQUID to mix VINTAGE Halo Opaque porcelain powder.
2. When the mixture loses optimum moisture, always add some VINTAGE Halo OPAQUE LIQUID for alteration.
3. Always use nylon brushes to build up porcelain mixture.
4. Tightly close the bottle cap after each use to avoid evaporation.
5. Do not use this product for any purposes other than specifically outlined in these instructions for use.

INDICATIONS FOR USE

Mixing liquid for VINTAGE Halo Opaque porcelain powder

PACKAGE

VINTAGE Halo OPAQUE LIQUID 50 mL / 500 mL

03:2019-03

Manufacturer

SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA

[EC REP] SHOFU DENTAL GmbH Am Brühl 17, 40878 Ratingen, Germany

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China

SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

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