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# VINTAGE MARGIN Porcelain Isolation Liquid

English

To achieve optimum results,  
please refer to these  
instructions.

### Description

Product Name	State	Ingredients
VINTAGE MARGIN Porcelain Isolation Liquid	Liquid	Alkyl siloxane

### Characteristics

Isolation liquid for preventing contact between porcelain and the plaster working model.

### Application

Before building-up VINTAGE MARGIN porcelain, apply VINTAGE MARGIN Porcelain Isolation Liquid to the surface of the plaster margin.  
If additional porcelain has to be built up, repeat this procedure.  
To supply the tip with liquid, place the cap on the rear button and press.  
When not in use, place the stick horizontally or vertically with the tip facing up, otherwise liquid might leak out.

### Caution

- Do not use, or keep near fire, as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperatures or direct sunlight.
- Keep it tightly closed if not in use.
- Do not use for purposes other than that described here.
- This product is for use by dental professionals only.

### Important

Do not touch with hands. Never allow this liquid to contact the skin or eyes. In case of skin contact, wipe off with cotton wool soaked in alcohol and rinse immediately with running water. In case of eye contact, rinse immediately with copious amounts of water and contact a doctor.  
If eruption or inflammation occurs, stop using this product and contact a doctor.

### Storage Conditions

Avoid direct sunlight or fire, seal the cap tightly and keep at room temperature.  
Keep this product away from persons other than dental professionals.

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