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VINTAGE Halo OPAQUE LIQUID-L

EN

VINTAGE Halo OPAQUE LIQUID-L is mixing liquid for VINTAGE Halo Opaque porcelain powder to keep porcelain mixture moist for longer time than VINTAGE Halo OPAQUE LIQUID, allowing users to work longer with VINTAGE Halo Opaque porcelain in optimal consistency.

DIRECTIONS FOR USE

- Use this liquid instead of VINTAGE Halo OPAQUE LIQUID.
- This liquid can be mixed with VINTAGE Halo OPAQUE LIQUID to obtain optimal consistency.
- Firing schedule is the same as when VINTAGE Halo OPAQUE LIQUID is used.
- Use VINTAGE MODELLING LIQUID for VINTAGE Halo Body porcelain.

NOTE ON USE

1. When the mixture loses optimum moisture, always add some VINTAGE Halo OPAQUE LIQUID for alteration.
2. Tightly close the bottle cap after each use to avoid evaporation.
3. Do not use this product for any purposes other than specifically outlined in these instructions for use.

INDICATIONS FOR USE

Mixing liquid for VINTAGE Halo Opaque porcelain powder

PACKAGE

VINTAGE Halo OPAQUE LIQUID-L 50 mL

03:2019-03

Manufacturer

SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA

[EC REP] SHOFU DENTAL GmbH Am Brühl 17, 40878 Ratingen, Germany

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China

SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

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