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VINTAGE Porcelain Isolation Liquid

English

To achieve optimum results,
please refer to these
instructions.

Description

| Product Name | State | Ingredients |
|------------------------------------|--------|----------------|
| VINTAGE Porcelain Isolation Liquid | Liquid | Alkyl siloxane |

Characteristics

Separating liquid for preventing contact between porcelain and the plaster working model.

Application

Before building-up the porcelain, apply 1-2 layers of VINTAGE Porcelain Isolation Liquid to the plaster with a brush. If additional porcelain has to be built up, repeat this procedure. A brush with natural bristles is recommended.

Caution

- Do not use, or keep near fire, as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperatures or direct sunlight.
- Keep it tightly closed if not in use.
- Do not use for purposes other than that described here.
- This product is for use by dental professionals only.

Important

Do not touch with hands. Never allow this liquid to contact the skin or eyes. In case of skin contact, wipe off with cotton wool soaked in alcohol and rinse immediately with running water. In case of eye contact, rinse immediately with copious amounts of water and contact a doctor. If eruption or inflammation occurs, stop using this product and contact a doctor.

Storage Conditions

Avoid direct sunlight or fire, seal the cap tightly and keep in a cool place. Keep this product away from persons other than dental professionals.

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EC [REP] SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684