EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03:2019-03
71398

SHOFU Composite Finishing Kit

EN **READ CAREFULLY BEFORE USE**

This kit is designed for reducing and finishing composite fillings, also interproximal and subgingival. It has six Dura-Green and six Dura-White Stones in CN1 and FL2 shapes. Dura-Green Stones will fast finish composite restorations. Dura-White Stones will fine finish composite restorations. Dura-Green and Dura-White Stones are autoclavable for reuse.

NOTES

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

- Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(NOTES)

●Never exceed the maximum permissible rotation speeds. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

(STERILIZATION)

●Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. ●Do not perform plasma sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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