Composite Polishing Kit

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03:2019-03
71399

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Dura-Green Stones	30,000 min⁻¹
Dura-White Stones	30,000 min⁻¹
CompoSite	20,000 min⁻¹
CompoSite Fine	20,000 min⁻¹

AUTOCLAVE (Table B)

Temperature	Time
134 °C/ 273 °F	3'00"
121 °C/ 250 °F	30'00"

DE MAXIMAL ZULÄSSIGE DREHZAHL
FR VITESSE MAXIMALE DE ROTATION AUTORISÉE
NL MAXIMAAL TOELAATBARE ROTATIESNELHEID
ES VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
SV HÖGSTA TILLÄTNA ROTATIONSHASTIGHET
PT VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

DE AUTOKLAV
FR AUTOCLAVE
NL AUTOCLAAF
ES AUTOCLAVE
IT AUTOCLAVE
SV AUTOKLAV
PT AUTOCLAVE

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EN

READ CAREFULLY BEFORE USE

This kit is designed for finishing and polishing all types of composite restorations. It includes two Dura-Green and two Dura-White Stones for finishing, six CompoSite for prepolishing and polishing and two CompoSite Fine (white band) Polishers for superpolishing. Dura-Green and Dura-White Stones, CompoSite and CompoSite Fine Polishers are autoclavable for reuse.

NOTES

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

- Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(NOTES)

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Dura-Green Stones and Dura-White Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization.
- CompoSite and CompoSite Fine: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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