 **Manufacturer**
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

CE 0123

EC REP
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03:2019-03
71397

Porcelain Veneer Kit

READ CAREFULLY BEFORE USE

EN

This kit is designed for making porcelain veneer placement. Kit has one Regular-grit and two Fine-grit FG Robot diamond points, three T&F Hybrid points, two Dura-White stones in FL2 and RE1 shapes, eight Ceramist Standard (for prepolishing), Ultra (for polishing) and Ultra II (for superpolishing) in knife, wheel and cup shapes. Robot Points, T&F Hybrid Points and Dura-White Stones will finish porcelain smoothly. Ceramist Polishers will polish and final polish porcelain veneers. Robot Points, T&F Hybrid Points, Dura-White Stones and Ceramist Polishers are autoclavable for reuse.

NOTES

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

- Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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- (NOTES)**
- Never exceed the maximum permissible rotation speeds.
 - Insert the shank completely into the chuck of the handpiece.
 - Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
 - Perform test run to check safety and smooth operation without run-out prior to use.
 - Avoid any use at acute angles or with excessive pressure.
 - Use local dust extractor and protective equipment while using.
 - Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
 - This product is intended for use by dental professionals only.
 - Do not use this product for any purposes other than the intended use.

- (STERILIZATION)**
- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization.
 - When using intraorally, sterilize before each use.
 - For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
 - Robot Points and T&F Hybrid Points: Do not immerse into chlorine disinfectant.
 - Dura-Green Stones and Dura-White Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization.
 - Ceramist: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
 - Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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