Brownie Greenie Supergreenie

04:2019-03
300030309

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
Am Brühl 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Cup FG, Floppie	15,000 min ⁻¹
Unmouted Wheel, Disk, Supergreenie PC2	20,000 min ⁻¹
Others	30,000 min ⁻¹

AUTOCLAVE (Table B)

Temperature	Time
134 °C/273 °F	3'00"
121 °C/250 °F	30'00"

DE MAXIMAL ZULÄSSIGE DREHZAH
FR VITESSE MAXIMALE DE ROTATION AUTORISÉE
NL MAXIMAAL TOELAATBARE ROTATIESNELHEID
ES VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
SV HÖGSTA TILLÄTNA ROTATIONSHASTIGHET
PT VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

DE AUTOKLAV
FR AUTOCLAVE
NL AUTOCLAAF
ES AUTOCLAVE
IT AUTOCLAVE
SV AUTOKLAV
PT AUTOCLAVE

EN

READ CAREFULLY BEFORE USE

Brownie, Greenie, Supergreenie silicone polishers are for polishing dental restorations. Brownies are for prepolishing, Greenies are for polishing and Supergreenies (with yellow banded shank) are for superpolishing.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points.
- Read all instructions on the back side carefully before use.

STERILIZATION

- Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

- Never exceed the maximum permissible rotation speeds.
- Use FG type with lower speed airtight handpiece. High speed air turbine units can break cups or points.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion.
- Use local dust extractor and protective equipment while using.
- Amalgam should be polished under running water to avoid heat generation.

Thoroughly remove amalgam dust. ● Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ● This product is intended for use by dental professionals only. ● Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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