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### EN

#### INDICATIONS FOR USE

Separating agent for easy removal of polymerized indirect composite resin from plaster models.

#### DIRECTIONS FOR USE

- Application: Shake the bottle well before use. Apply CERAMAGE SEP evenly to the entire inner surface of the cavity, opposing tooth, and contact areas of proximal teeth. After drying, apply composite resin and light cure. If the separation from a model seems to be difficult, apply again.
- Removal / cleaning: After separating composite resin from the model, clean with a toothbrush, ultrasonic cleaner, steam cleaner, or sandblast. (If the restoration is cemented with residue of separating agent, proper bonding cannot be obtained.)

#### NOTE ON USE

1. If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
2. Do not touch the material with bare hands. This product should not come in contact with skin or eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
3. Avoid any source of ignition since this material is flammable.
4. Use this material in well ventilated area (several appropriate ventilations per hour are recommended).
5. Tightly close the cap immediately after each use.
6. Do not use this product for any purposes other than specifically outlined in these instructions for use.
7. This product is intended for use by dental professionals only.

#### COMPOSITION

Ethanol, Glycerin, Polyethylene glycol

#### STORAGE

- Store at room temperature (1-30 °C / 34-86 °F). Keep away from direct sunlight and any source of ignition.

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