Acrylic Contouring & Finishing Kit

02:2017-10
71067

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

CONFIGURATION

	Shape	PN	ISO No.
AcryPoint	Dark Grey	BP1	0426 658 105 499 532 100
		PC2	0423 658 104 292 532 060
	Brown	BP1	0427 658 105 499 522 100
		PC2	0424 658 104 292 522 060
	Light Grey	BP1	0428 658 105 499 512 100
		PC2	0425 658 104 292 512 060
Robot Carbide Cutter	SH251E	0620	500 104 274 190 060
	SH79E	0616	500 104 194 190 040
	SH139E	0635	500 104 289 190 023
	SH33	0667	500 104 168 007 009

FR CONFIGURATION
ES CONFIGURACIÓN
PT CONFIGURAÇÃO

EN

READ CAREFULLY BEFORE USE

- To adjust acrylic resins, use Robot Carbide Cutters or AcryPoint Coarse (dark grey)
- To finish, use AcryPoint Medium (Brown)
- When polishing, use fine AcryPoints (light grey) to high-lustre polish acrylic.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

MAXIMUM PERMISSIBLE ROTATION SPEED

AcryPoint	15,000 min ⁻¹
Robot Carbide Cutter	30,000 min ⁻¹

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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