## Porcelain Adjustment Kit

**Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

02:2017-05  
70493

READ CAREFULLY BEFORE USE

EN

This kit is designed for making chairside adjustments of porcelain restorations and (polishing adjusted surfaces to a glaze-like finish. The kit has three Dura-White Stones for adjusting, three Ceramisté Standard Polishers for prepolishing, three Ultra (yellow band) Polishers for polishing and three Ultra II (white band) Polishers for superpolishing. Dura-White Stones and Ceramisté Polishers are autoclavable for reuse.

### PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

### STERILIZATION

- When necessary, perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When necessary, remove any contaminant with cleaning liquid or disinfectant.
- Dura-White Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization.
- Ceramisté: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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