 **Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

 **CE 0123**

**EC REP**  
SHOFU DENTAL GmbH  
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

03:2019-03  
300030279

## MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

FG Cup	<b>15,000 min<sup>-1</sup></b>
Others	<b>30,000 min<sup>-1</sup></b>

## AUTOCLAVE (Table B)

Temperature	Time
<b>134 °C/273 °F</b>	<b>3'00"</b>
<b>121 °C/250 °F</b>	<b>30'00"</b>

DE MAXIMAL ZULÄSSIGE DREHZAH  
FR VITESSE MAXIMALE DE ROTATION AUTORISÉE  
NL MAXIMAAL TOELAATBARE ROTATIESNELHEID  
ES VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN  
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA  
SV HÖGSTA TILLÄTNA ROTATIONSHASTIGHET  
PT VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

DE AUTOKLAV  
FR AUTOCLAVE  
NL AUTOCLAAF  
ES AUTOCLAVE  
IT AUTOCLAVE  
SV AUTOKLAV  
PT AUTOCLAVE

## Amalgam Polishing Kit

**EN** **READ CAREFULLY BEFORE USE**

This kit is designed for polishing and superpolishing amalgam and precious alloys. It includes three each Brownie cups and Mini-Points, two each Greenie cups and Mini-Points, one each Supergreenie (yellow band) cup and Mini-Point, Brownie, Greenie and Supergreenie Polishers are autoclavable for reuse.

### NOTES

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

### STERILIZATION

- Perform autoclave under the conditions described in Table B.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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## (NOTES)

●Never exceed the maximum permissible rotation speeds. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●For polishing amalgam, use under running water to avoid generating heat. Ensure that polishing dust is suctioned completely. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

## (STERILIZATION)

●Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. ●Do not perform dry heat sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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