Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

Ceramage Finishing & Polishing Kit

02:2017-05
70945

[EC REP]
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EN

This assortment is recommended for systematic finishing and polishing of highly ceramic filled light curing composite materials such as CERAMAGE. It includes one Robot Carbide Cutter (Fissure Bur), three Dura-Green Stones, three CompoMaster Coarse (yellow polishers) and three CompoMaster (white & yellow bands). First, use Robot Carbide Cutter for finishing of the interproximal and occlusal anatomical details and fissures. Then use Dura-Green Stones for grinding and contouring the composite surface. Next use CompoMaster Coarse for finishing and polishing and CompoMaster for super polishing. CompoMaster Coarse and CompoMaster are durable and filled with a high density of pure diamond particles. They are designed for quick finishing and polishing of high ceramic filled composite restorations to provide a smooth surface. All polishers are autoclavable for reuse. Also, by using DURA-POLISH and DURA-POLISH DIA polishing wax sold separately, higher shiny surface gloss can be achieved.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

- When necessary, perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization.
- When necessary, remove any contaminants with cleaning liquid or disinfectant, then sterilize.
- Robot Carbide Cutter: Do not immerse into chlorine disinfectant.
- Dura-Green Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization.
- CompoMaster/CompoMaster Coarse: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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