# Ceramisté

**Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

CE 0123

**EC REP**

SHOFU DENTAL GmbH  
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

03:2019-03  
300030292

**EN**

**READ CAREFULLY BEFORE USE**

Ceramisté silicone polishers are for polishing porcelain and enamel. Ceramisté Standard polishers are for prepolishing. Ceramisté Ultra polishers (yellow band) are for polishing and Ceramisté Ultra II polishers (white band) are for superpolishing.

### PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points.
- Read all instructions on the back side carefully before use.

### STERILIZATION

- Perform autoclave under the conditions described in Table B.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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## (CAUTIONS)

- Never exceed the maximum permissible rotation speeds.
- Use FG type with lower speed airtight handpiece. High speed air turbine units can break cups or points.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

## (STERILIZATION)

- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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