 **Manufacturer**
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

 0123

[EC REP]
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

CompoMaster Finishing & Polishing Kit

03:2019-03
71407

[EN] **[READ CAREFULLY BEFORE USE]**

This assortment is recommended for systematic finishing and polishing of composite restorations. It includes four T&F Hybrid Points, three CompoMaster Coarse Polishers (yellow), and three CompoMaster Polishers (white and yellow bands). First use T&F Hybrid Points for grinding and contouring the composite restoration. Then use CompoMaster Coarse for finishing and polishing and CompoMaster for superpolishing. The CompoMaster, which are filled with a high density of pure diamond particles, are durable and designed for quick finishing and polishing of composite restorations to provide a smooth surface. All polishers are autoclavable for reuse. When additional superpolishing is required, use DirectDia Paste or SuperBuff polishing discs.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

STERILIZATION

Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Use T&F Hybrid Points with water spray.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- T&F Hybrid Points: Do not immerse into chlorine disinfectant.
- CompoMaster: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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