Dura-Green DIA

EN Instructions for use

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READ CAREFULLY BEFORE USE

Dura-Green DIA is a diamond impregnated abrasive with glass binder. SHOFU ADJUSTER is available as an accessory to dress Dura-Green DIA.

SHAPE

HP: CY5, FL3, IC5, KN7, RD3, TC2, TC4, WH6
CA: FL3, RD3, TC2
FG: FL3, RD2

INDICATIONS

Contouring and finishing of indirect restorations

- Porcelain
- Zirconia
- Lithium Disilicate etc.

PRECAUTIONS

1. This product is intended for use by dental professionals only.
2. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.

DIRECTIONS FOR USE

1. When using intraorally, sterilize this product before use.
 - Sterilization method: autoclave this product for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
 - This product should be sterilized or disinfected by any of the following methods; chemiclave, autoclave, disinfectant liquid. Sterilize or disinfect according to each manufacturer's Instructions for use. For disinfection with chloride disinfectant liquid (sodium hypochlorite and benzalkonium chloride solution), do not immerse this product as it may cause corrosion or the working part may come off due to degradation to bonding between the head and the shank.

2. Mount this product on a dental handpiece (dental electric engine, micro motor, high-speed lathe etc.).
 - Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
 - Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
 - Perform test run to check safety and smooth operation without run-out prior to use.
3. Use following the conventional manner.
 - Never exceed the maximum permissible rotating speed.

Point	Maximum permissible rotating speed
	30,000 min ⁻¹

- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
 - Use local dust extractor, protective mask, etc while finishing to avoid adverse effect of dust to the human body.
 - Use protective equipment such as glasses while using.
 - Use SHOFU ADJUSTER to remove clogging of the debris or to adjust the abrasive to the desired size.
4. For reuse, remove any contaminants using cleaning liquid or disinfectant and sterilize.

STORAGE

Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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