Dura-Green Stones Fine

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

02:2017-12
SSDM000032

EN

READ CAREFULLY BEFORE USE

FR

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Dura-Green Stones Fine are impregnated with Silicon Carbide grains in a glass binder. Dura-Green Stones Fine are intended for contouring and finishing porcelain and composite.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

MAXIMUM PERMISSIBLE ROTATION SPEED

Dura-Green Stones Fine	30,000 min ⁻¹
------------------------	--------------------------

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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