Gold Polishing Kit

Manufacturer
SHOFU INC.
 11 Kamitakamatsu-cho, Fukuine,
 Higashiyama-ku, Kyoto 605-0983, Japan
 www.shofu.com

SHOFU DENTAL CORPORATION
 1225 Stone Drive, San Marcos,
 California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
 10 Science Park Road, #03-12,
 The Alpha, Science Park II, Singapore 117684

02:2017-04
 30400

READ CAREFULLY BEFORE USE

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ES

PT

This kit is designed for finishing and superpolishing cast gold/precious alloys. It includes six Brownie, three Greenie and three Supergreenie Polishers. Brownies for prepolishing, Greenies for polishing and Supergreenies (with yellow banded shank) for superpolishing. Brownie, Greenie and Supergreenie Polishers are autoclavable for reuse.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

STERILIZATION

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization (Table B).
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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