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GBCT000009 01:2017-07

Read "NOTE ON USE" in these Instructions for Use before use

**This product is exclusively for dental lab light-curing unit.  
Do not use for porcelain furnace.**

## SHOFU Post Set / Post Stand P for light-curing Instructions for Use

### INTENDED USE

For stable placement of crown restorations in dental lab light-curing unit (Solidilite V etc.)

### FEATURES

#### SHOFU Post Set for light-curing

With 5 different shapes available, suitable shape can be selected for each restoration in order to place the restoration stably in a dental lab light-curing unit. Posterior post B (for molar) can also be used to stand post core. Free post can be grinded to create a preferred shape.

#### SHOFU Post Stand P for light-curing

Two crossing central ditches extend down to the bottom of the post stand, which allows operators to stand a Dowel Pin easily. Also, twin pins and multiple Dowel Pins can be placed in this post stand.

### DIRECTIONS FOR USE

#### 1. Placement of posts


Insert the post shank(s) into the circle hole(s). Anterior post A, Posterior post A and Posterior post B have decentralized shanks, which enable the heads to rotate freely in order to adjust the space between restorations on one stand.

#### 2. Placement of Dowel Pin mounted model

Insert the Dowel Pin into the ditch. Refer to the figures below and select the most suitable ditch to stand the pin.


A. Single pin

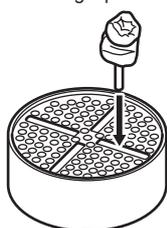B. 2 single pins

C. Twin pin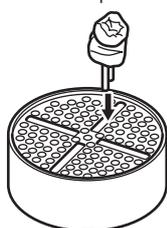

#### 3. Grinding of free post

Carbide burs are recommended to grind free post since it incorporates glass fiber. Avoid grinding with carborundum point in order to avoid roughening the surface. Other posts can also be grinded to create desirable shape.

### NOTE ON USE


1. This product is flammable. Do not use and store near any source of fire.
2. Do not use this product for any purpose other than specifically outlined in [INTENDED USE] in these Instructions for Use.
3. This product is intended for use by dental professionals only.
4. When grinding this material;
  - Use protective eye glasses etc. in order to avoid damages to eyes.
  - Use local dust collector and dust mask to prevent adverse effects on human body.

### STORAGE

Store this product at room temperature.

### CONFIGURATION

1. SHOFU Post Set for light-curing (5 shapes, 3 each)

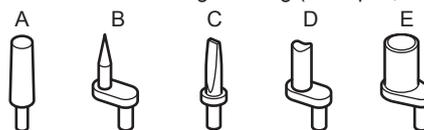A Free post

Can be grinded freely to create desirable shape.

B Anterior post A (cone shape)

Mainly for anterior copings. Its decentralized shank and freely rotating head allow operators to place multiple posts in one post stand while keeping necessary space between the restorations.

C Anterior post B (wedge shaped)

Mainly for anterior copings. Its wedge shaped tip ensures stable placement of restoration.

D Posterior post A (for premolar)

Mainly for premolars.

E Posterior post B (for molar)

Mainly for molars. Post core can also be inserted into this tube shape post.

2. SHOFU Post Stand P for light-curing

Part name	Quantity
Post Stand P	1 pc

Material: Nylon (containing glass fiber 20%)

Operating temperature limit: Approximately 200°C

\*Yellow discoloration might occur due to aging and heat.

	Outer dimension (mm)
Free post	φ5x23
Anterior post A (cone shape)	W5xD10xH23
Anterior post B (wedge shaped)	φ5x23
Posterior post A (for premolar)	W5xD10xH21
Posterior post B (for molar)	W8xD11.5xH21
Post Stand P	φ60x23.5

Manufacturer  
**SHOFU INC.**, 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan  
**SHOFU DENTAL CORPORATION**, 1225 Stone Drive, San Marcos, California 92078-4059, USA  
**SHOFU DENTAL GMBH**, Am Brühl 17, 40878 Ratingen, Germany  
**SHOFU DENTAL TRADING (SHANGHAI) CO., LTD.**, No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China  
**SHOFU DENTAL ASIA-PACIFIC PTE. LTD.**, 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684  
[www.shofu.com](http://www.shofu.com)

GBCT000009 02:2017-07