Manufacturer

SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

03:2019-02  
300030283

EN

READ CAREFULLY  
BEFORE USE

SoftCut Polishers are used for polishing fragile porcelain veneers, inlays and onlays. Use PA (green band) for polishing after rough finishing (for example with Dura-Green Stones) Use PB (blue band) for polishing after fine finishing (for example with Dura-White Stones). Adjust polishing performance by varying the pressure: the lighter the pressure, the higher the polish. Start with normal to light pressure to adjust polishing performance.

#### PRECAUTIONS

- Never exceed the maximum permissible rotation speeds.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

#### MAXIMUM PERMISSIBLE ROTATION SPEED

Polishers	20,000 min <sup>-1</sup>
Unmounted Wheels	15,000 min <sup>-1</sup>

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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