PRODUCT DESCRIPTION

Super-Snap SuperBuff is a paste-impregnated disposable disk designed for high gloss polishing of composite restorations. Before use, slightly moisten the SuperBuff Disk with water. During use, it develops a polishing compound on the surface which leads to a high gloss in a short time.

INDICATIONS FOR USE

Polishing of composite restorations

PRECAUTIONS AND WARNINGS

1. Super-Snap SuperBuff disks are designed for single use only. Discard the disks after use in accordance with the disposal protocol of your healthcare facility.
2. Do not use damaged or defective disk or mandrel, i.e. deformation, dirt, corrosion.
3. Use protective mask and eyewear when operating handpieces.
4. Do not use this product for any purposes other than specifically outlined in these instructions for use.
5. The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Mount a SuperBuff disk on a dedicated metal mandrel, Super-Snap Mandrel.
 - Sterilize the metal mandrel before each use.
2. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
3. Wet the surface to be polished and moisten the SuperBuff disk with water to generate creamy white polishing paste.
4. Apply uniform and feather light pressure for polishing.
 - To polish occlusal surface, use the flat edge of the disk.
 - Do not use intermittent water. It will dilute polishing effect.
 - Avoid exceeding maximum permissible speed.
5. Rinse patient's mouth after use, when necessary.
6. After using dedicated metal mandrel, remove any contaminants immediately, disinfect and sterilize. Dispose the Super-Snap SuperBuff disks after each use.

Super-Snap SuperBuff	15,000 min ⁻¹
----------------------	--------------------------

STERILIZATION AND STORAGE

Super-Snap SuperBuff and Super-Snap Metal Mandrel should be stored away from direct sunlight, moisture and corrosive chemicals/vapor.

Remove any contaminants using suitable cleaning liquid or disinfectant and perform autoclave to sterilize the metal mandrel.

- Autoclave : Metal mandrel

Temperature	Time
134 °C/273 °F	3'00"
121 °C/250 °F	30'00"

Follow manufacturer's instructions for use of cleaning liquid, disinfectant or sterilizer. Chloride disinfectant should be avoided.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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