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VINTAGE AL / ZR OPAQUE LINER LIQUID

English

To achieve optimum results,
please refer to these instructions.

Description

Product Name	State	Ingredients
VINTAGE AL/ZR OPAQUE LINER LIQUID	Liquid	Glycerol, propylene glycol

Characteristics

Modelling liquid for VINTAGE AL & VINTAGE ZR OPAQUE LINER.

Application

Use for mixing VINTAGE AL & VINTAGE ZR OPAQUE LINER and adjusting handling properties.

Please note

Do not use with other products besides VINTAGE AL & VINTAGE ZR OPAQUE LINER.

Caution

- Do not use, or keep near fire, as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperatures or direct sunlight.
- Keep it tightly closed if not in use.
- Do not use for purposes other than that described here.
- This product is for use by dental professionals only.

Important

Do not touch with hands. Never allow this liquid to contact the skin or eyes. In case of skin contact, wipe off with cotton wool soaked in alcohol and rinse immediately with running water. In case of eye contact, rinse immediately with copious amounts of water and contact a doctor.

If eruption or inflammation occurs, stop using this product and contact a doctor.

Storage Conditions

Avoid direct sunlight or fire, seal the cap tightly and keep at room temperature.
Keep this product away from persons other than dental professionals.

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VINTAGE AL/ZR OPAQUE LINER LIQUID 3mL

Manufacturer
SHOFU INC, 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA
[EC REP] SHOFU DENTAL GmbH Am Brühl 17, 40878 Ratingen, Germany
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
www.shofu.com

03:2019-03 71653